

Indiana State University
Nursing Program
SAM II Instructions

The laptop computer needs to remain connected to the SAM II. If laptop has been disconnected or turned on out of sequence, see ** below to insure speakers are set up correctly.

The sequence of starting the system is crucial.

- Turn on SAM II by pushing the square button in / on
- Turn on Speakers (toggle switch on back of left speaker (when facing the front of the cart). (the I on the toggle should be “in”)
- Turn on laptop with power button
- Open the Cardionics program
- Select sound(s) as needed
- Click on Play
- Click on the circle on the diagram of SAM II for which external sound is desired

*****If unable to hear sound from external speakers, check the following:***

- Move mouse to bottom of screen for taskbar to appear
- Click on the Windows logo button that appears on left of taskbar
- Click on Control Panel
- Click on Hardware and Sound
- Click on Sound
 - There should be 3 items listed:
 - Speaker
 - 3-USB Speaker (should be the default speaker)
 - If the 3-USB speaker is NOT the default speaker, click on it and then click on Set as Default
 - Digital Tuner
 - Click on the 3-USB speaker, then click on Configure
 - Make sure that 7.1 Surround Sound is selected
 - Click on 7.1 Surround if it is not selected and then “click on through” the 7.1 Surround Sound setup.
 - Close out of the Sound area
- Re-try opening the Cardionics program and selecting sounds again.